

Gutta Percha Points

Manufactured for:
US Endodontics, LLC
2809 W. Walnut St,
Johnson City, TN 37604, USA
Tel: +1-423-283-0047
regulatoryaffairs@us-endo.com
www.edgeendo.com



Manufacturer

META BIOMED CO., LTD.

270, Osongsaengmyeong 1-ro,
Osong-eup, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, Republic of Korea

Tel +82-43-216-0433

Fax +82-43-217-1988

E-Mail info@meta-biomed.com

www.meta-biomed.com

EC REP

Meta Biomed Europe GmbH

Wiesenstr. 35, 45473 Mülheim an der Ruhr,
Germany

Tel +49-208-30991910

Fax +49-208-30991999



IFU-MBM-GPP Rev.A 04-05-2023

Made in Korea

Gutta percha points are root canal obturation material. It is used to fill the cleaned, shaped and irrigated root canal space.

- Time saving with easy filling
- For faster and easier root canal obturation
- Variable tapers & sizes
- Length marking available
- Excellent radiopacity
- Long shelf life and freshness assured
- Suitable for vertical and lateral condensation

INDICATION AND PURPOSE

- ① Gutta percha point is an integral material in promoting periapical healing and preventing disease progression.
- ② Endodontic therapy system using the gutta percha point obturated impermeable to liquid and bacteria

INSTRUCTION FOR USE

- ① Prepare root canal by file.
- ② After clean and dry the canal with EDTA solution, apply the sealer into the root canal wall by paper points.
- ③ Select master gutta percha point matched in prepared root canal.
- ④ Irrigate a gutta percha point with 5.25% NaOCl disinfecting solution.
- ⑤ Insert mater points in to the root canal as working length.
- ⑥ Insert accessory points into the root by finger spreader.
- ⑦ Confirm a filling by the x-ray.

CAUTIONS

- ① Do not use this product for any patients with a history of hypersensitivity reactions to it, such as a rash, dermatitis, etc.
- ② Some patients have an allergic reaction to Trans 1,4 Polyisoprene materials and/or even experience hypersensitivity. Do not use this product for them

- ③ This product is intended only for use for the tissue inside the mouth. Be careful to prevent it from coming into contact with eyes. If material has come into contact with eyes or skin, wash immediately with plenty of water and then consult a doctor.
- ④ Non-specialists are prohibited from using this product. Do not use this product for any purposes other than its intended purposes.
- ⑤ Do not prescribe an internal medicine
- ⑥ Should be careful to avoid overfilling or dead space
- ⑦ Be sure to take measures for the protection of the respiratory tract before
- ⑧ Always close the slide lid after use space

STORAGE

Store at room temperature of 1℃~30℃ (33°F ~86°F)

SHELF LIFE

3 years form manufacture date

Available Size

Type	Sizes
Standardized (ISO Specifications)	10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80
Non-Standardized (Accessory Sizes)	06T#15 (Verifile), 06T#20 (Small), 06T#25 (Primary), 06T#35 (Medium), 06T#45 (Large), F1/F2/F3/F4/F5
Special Taper (.04, .06)	15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80

COMPONENTS

- Available packaging

- ① Sliding type dispenser - Code: P.71/
Regular packing points: 60/120 points

SYMBOLS

Use By

Batch Code

Authorized representative in the EC

Caution

Do not reuse

Manufacturer

Date of manufacture

Keep away from sunlight

Temperature limitation

CE 1639 CE- ark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

Rx Only Caution : Federal law restricts this device to sale by or on the order of a dentist



EDGEENDO®