

# Absorbent Paper Points

Manufactured for:  
US Endodontics, LLC  
2809 W. Walnut St,  
Johnson City, TN 37604, USA  
Tel: +1-423-283-0047  
regulatoryaffairs@us-endo.com  
www.edgeendo.com



Manufacturer

**META BIOMED CO., LTD.**  
270, Osongsaengmyeong 1-ro,  
Osong-eup, Heungdeok-gu, Cheongju-si,  
Chungcheongbuk-do, Republic of Korea  
Tel +82-43-216-0433  
Fax +82-43-217-1988  
E-Mail info@meta-biomed.com  
[www.meta-biomed.com](http://www.meta-biomed.com)

**EC REP**

**Meta Biomed Europe GmbH**

Wiesenstr. 35, 45473 Mülheim an der Ruhr,  
Germany

Tel +49-208-30991910

Fax +49-208-30991999



IFU-MBM-PP Rev.A 04-05-2023

Made in Korea

"Hand"rolled for extra absorbency

- Highly absorbent
- Length marked available
- Variety sizes and packages available
- Color coded

## INDICATION AND PURPOSE

Paper points are used to dry the root canal before obturation and to clean the root canals that are to be sealed in endodontic treatments.

## INSTRUCTION FOR USE

1. Measure the length of root canal
2. The formation of the root canal of the same size as the last file to select
3. The root canal using paper points to dry
4. About 10 ~ 15 points is used per a root canal
5. Sealer or cement is applied by using paper points

## CAUTIONS

1. Should be use sterilized paper points
2. Do not use bent or broken points
3. Should be careful to avoid overfilling or dead space when applied sealer or cement
4. Use it immediately when applied sealer or cement
5. After using the paper points, should be kept in a cool place
6. Do not expose to direct sunlight
7. Do not reuse to prevent cross infections. (Secondary infections such as pathogens and viruses caused by saliva and blood)

## STORAGE

Store at room temperature of 1°C~30°C(33°F ~86°F)

## SHELF LIFE

5 years form manufacture date

## AVAILABLE SIZE

Type	Sizes
Standardized (ISO Specifications)	10, 15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80
Non-Standardized (Accessory Sizes)	06T#15 (Verifile), 06T#20 (Small), 06T#25 (Primary), 06T#35 (Medium), 06T#45 (Large), Protaper Size (F1~F5)
Special Taper (.04, .06)	15, 20, 25, 30, 35, 40, 45, 50, 55, 60, 70, 80

## COMPONENTS

- Available packaging

1. Sliding type dispenser - Code:  
P.71/Regular packing points: 60/200 points

## SYMBOLS

Use By

Batch Code

Sterilization using irradiation

Authorized representative in the EC

Consult Instruction For Use

Caution

Do not reuse

Do not resterilize

Manufacturer

Date of manufacture

Keep away from sunlight

Temperature limitation

CE- Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC

**Rx Only** Caution : Federal law restricts this device to sale by or on the order of a dentist



**EDGEENDO®**